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# Content

How to operate the B-Cure Laser device	27
Treatment dosage	31
Recommended treatment points	32
<b>Clinical areas and treatment applications</b>	<b>33</b>
• Chronic neck pain	34
• Chronic back pain	36
• Carpal tunnel syndrome (CTS)	38
• Knee pain	40
• Myofascial pain syndrome (MPS) / Fibromyalgia	42
• Temporomandibular joint disorder	44
• Wounds, pressure sores, diabetic sores, ulcers and hard to heal wounds	46
The B-Cure Laser adjustable stand	47

## How to operate the B-Cure Laser device

### Switch the device on

To switch the device on, press the round-shaped power button at the center for 3 seconds. The numbers 00:00 will appear on the screen.

## Set the treatment time

Press "+" at the right of the elongated button to increase the time by 30 seconds intervals.

Press "-" at the left of the elongated button to decrease the time by 30 seconds intervals.

## Activate the laser beam

To operate the laser beam in automatic mode, hold the operation button (on the left, banana shape) (1) for 2-3 seconds and – at the same time - press the round-shaped power button at the center of the device (2).

Then release both buttons **SIMULTANEOUSLY** (3).

The device is now operating in automatic mode (continuously).

Place the device gently in full contact with the skin. Hold the device on the skin steadily and motionless during the whole duration of the treatment. Do not treat on top of clothes or bandaging.

To switch off the device, press the main power switch until a beep sounds. Release the button and the display will go blank.

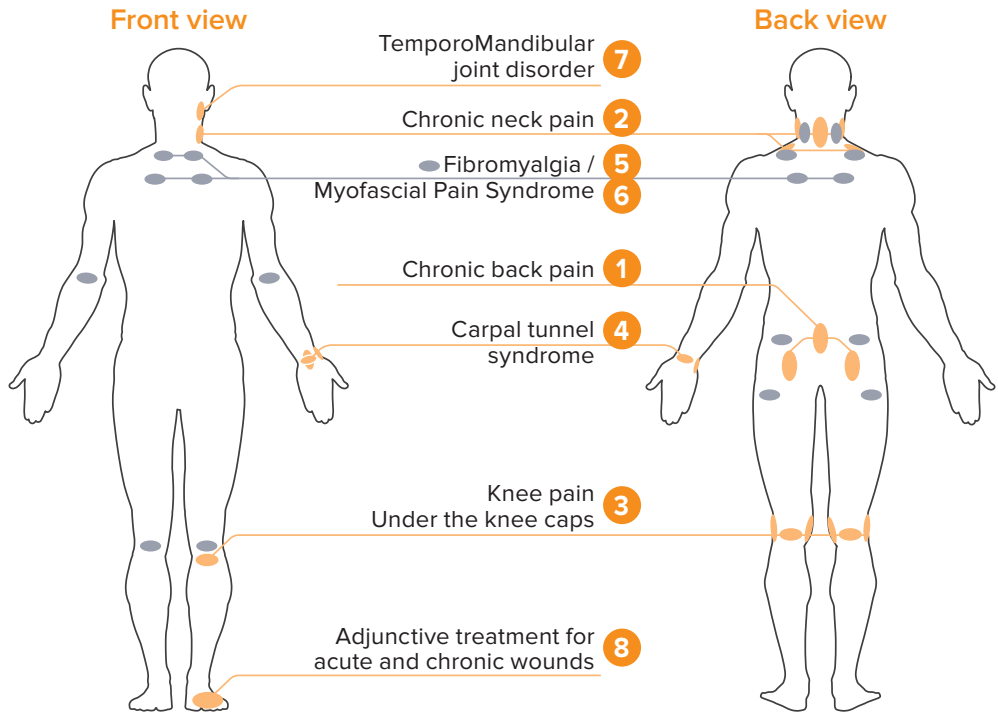
**⚠ Never aim the laser beam at the eyes.**

## Treatment Dosages

Start with a 1.5 minute dosage for each treatment point and increase the dosage gradually within 24 hours until you reach 6 min.

Treat twice a day, consistently. If you experience increased pain following treatment, stop the treatment for 24 hours then start again with a reduced dosage. You may increase the dosage to 4 treatments a day. In case of acute pain or trauma, you may apply a dosage of 6 minutes from the first treatment.

# Recommended Treatment Points

You may treat any aching point on the body, except the eyes.

# Clinical Areas and Treatment Applications (partial list):

Simultaneous treatment for pain relief, orthopedic conditions, wound healing and reduction of swelling and inflammation.

## Pain and injury of bones, tissue and orthopedic conditions

- Back pain: lower back, upper back, neck, and muscle pain
- Joints: knees, hips, shoulders, fingers, osteoarthritis
- Rheumatoid arthritis and/or chronic tendon inflammation
- Heel spurs and sprains
- Tennis elbow
- Achilles tendon inflammation
- Carpal tunnel syndrome (CTS)
- Myofascial pain syndrome

## Medical dermatology

- Wound healing
- Fresh scars and burns
- Promotes improved and more esthetic healing of surgical incisions

## Acupuncture treatments

# Neck

Treat the following locations:


- a. Lengthwise, along the vertebra.
- b. Treat each side of the neck - lengthwise, under the ear lobe, exactly at the indentation behind the jaw.

Three (3) treatment points, **18 minutes** for each treatment session in total.

You may also treat two additional points close to the bottom of the neck and toward the shoulders.

Treat 2-4 times a day.

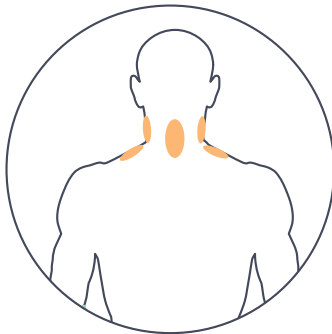Back view


# Lower back

Treat the following locations:

- a. Along the spine, lengthwise.
- b. Find the sensitive or sore points on the buttocks and treat both sides.

Treat 2-4 times a day, **6 minutes** on each sore point.

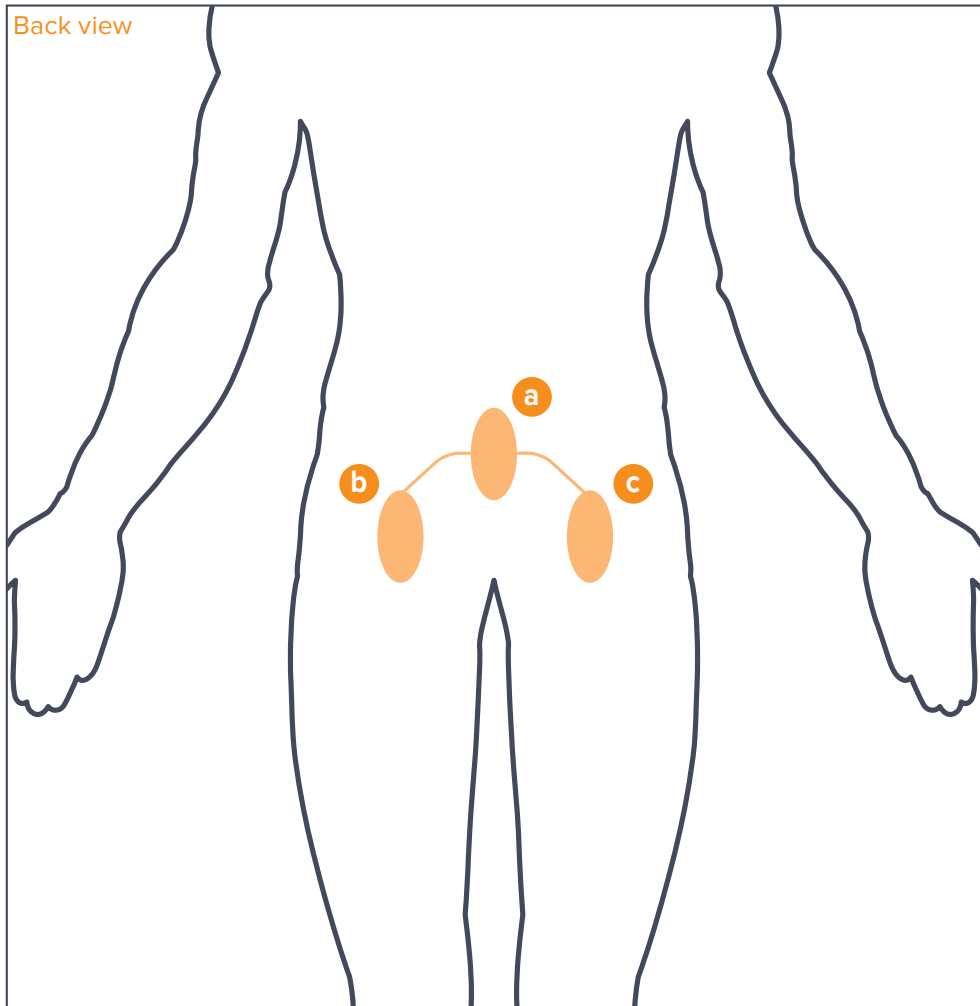Back view

# Carpal tunnel syndrome

Treat the following locations:

- a. On the inside, along the width of the joint – on the crease.
- b. On the inside, perpendicular to the previous point, lengthwise.
- c. Treat the length of the thumb, along the pad.
- d. Along the wrist bone, at the base of the hand, below the thumb.
- e. Exactly opposite to the previous point, below the “pinky” finger.

Five (5) treatment points, **30 minutes** for each treatment session in total.

Treat 2-4 times a day.

Front view

# Knees

Treat the following locations:


- a. On the back of the knee, along the inner fold.
- b. On the left side of the knee, lengthwise.
- c. On the right side of the knee, lengthwise.

Three (3) treatment points of 4.5 cm<sup>2</sup>, **18 minutes** for each treatment session in total.


Treat 2-4 times a day.

You may also treat the front of the knee, under the kneecap.

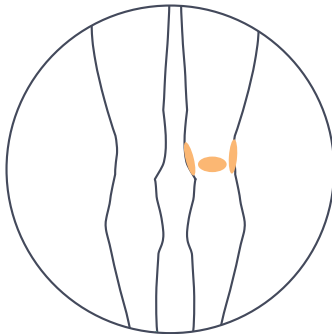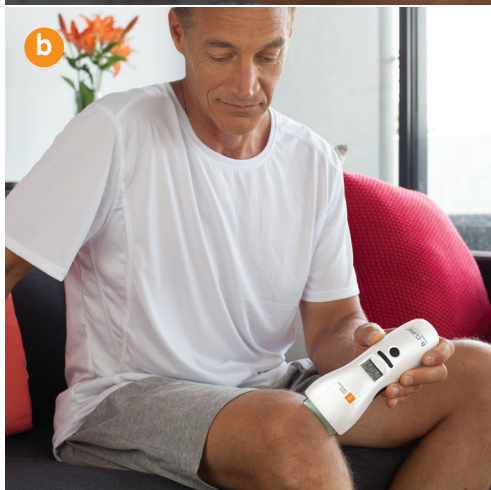Back view



# Myofascial pain syndrome/ Fibromyalgia

Treat for **6 minutes** each aching area, twice a day. In the illustration you may find suggestions for common trigger points.

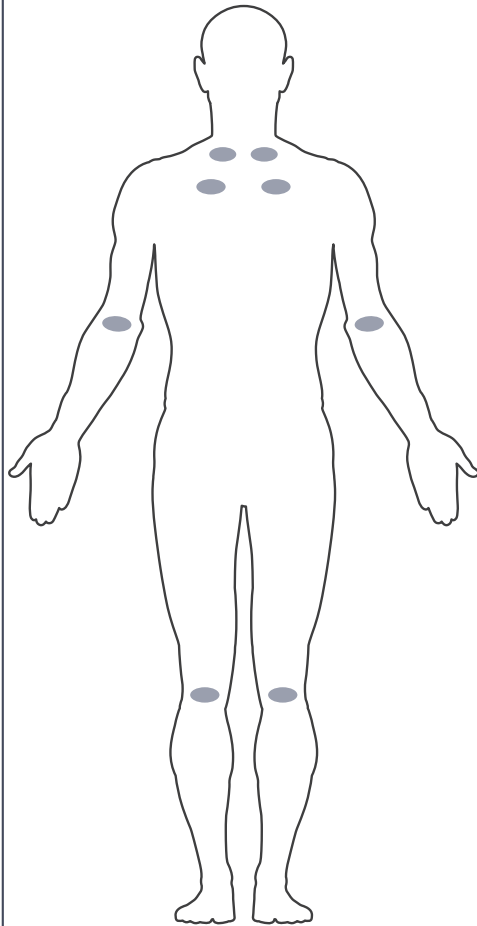Front view

Back view
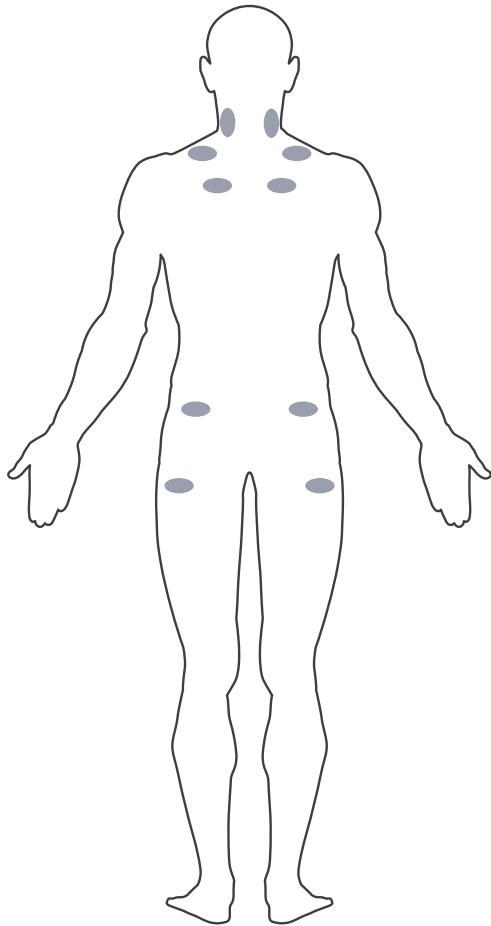
# Temporomandibular joint disorder

Position the device according to the direction of the beam recommended in the illustration: **6 minutes horizontally** and **6 minutes vertically**.

**2 treatment points, 12 minutes** for each treatment session in total.

Treat 2-4 times daily.

Front view

## Fresh wounds, pressure sores, diabetic sores, ulcers and hard to heal wounds

Set treatment time to **6 minutes**. Activate the laser beam and hold the device about 1 cm above the wound, motionless, until the termination of the set treatment time. If the area of the wound exceeds the size of the laser beam area ( $1 \times 4.5 \text{ cm}^2$ ), set the stop watch again to **6 minutes** and treat the adjacent treatment point. Repeat until the whole surface area of the wound has been covered.

The B-Cure Laser in the treatment of diabetic wounds and hard to heal wounds is in addition to standard therapies and is not intended to be a replacement.

## The B-Cure Laser adjustable stand

It is highly recommended to use the adjustable stand, for a more comfortable treatment.

Stabilize the device and the stand and treat the area, keeping as still as possible, so that the laser beam can penetrate deep into the tissues.

## B-Cure Laser

**Proizvajalec: Biocare Enterprise Limited**

Flat 1 A, 9/F, Brill Plaza, No.84, Tokwawan Road, Kowloon, Hong Kong

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Bd General Wahis 53 1030 Brussels, Belgija

**Obelis UK Ltd.**

Sandford Gate, Oxford, OX4 6LB, United Kingdom

**B-CURE**<sup>®</sup>  
LASER

GOOD  
ENERGIES<sup>®</sup>